Vinyl polysiloxane impression material  
DIN EN ISO 4823

**KETTENBACH DENTAL**


Simply intelligent

# SILGINAT®

**INSTRUCTIONS FOR USE** / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing /  
Инструкция по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullanım Kılavuzu /  
Instruções de uso / Instrukcja użycia / Інструкція для застосування / 取扱説明書 / تعليمات الاستخدام

# CONTENT

Inhalt / Contenu / Inhoud / содержимое /  
Contenuto / Contenido / İçerik / Conteúdo /  
Treść / Зміст / 目次 / الفهرس

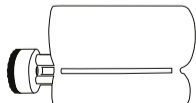Deutsch .....	4
English.....	10
Français.....	16
Nederlands .....	23
Русский.....	29
Italiano .....	36
Español.....	42
Türkçe .....	49
Português .....	55
Polski.....	61
Українська.....	68
日本語.....	76
83 .....	عربي

Vinyl polysiloxane impression material  
DIN EN ISO 4823

**Silginat<sup>®</sup>**  
**Silginat<sup>®</sup> Strawberry**

**For professional use only.**

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## Product Description

Alginate Alternative: Silginat® and Silginat® Strawberry are addition-curing, elastomeric impression materials intended to be used as an alternative to traditional alginate materials (e.g., for anatomical impressions). Available in 1:1 50 ml cartridges suitable for the Kettenbach Applyfix 4 dispensing gun and in 5:1 380 ml cartridges for use in most commercially available other automatic dispensing and mixing machines, e.g., Sympress.

## Indications / Techniques

Silginat® and Silginat® Strawberry are useful as alginate replacements for making impressions for:

- Opposing dentitions
- Fabricating temporary crowns and bridges
- Anatomical models
- Fabricating simple removable prosthetic restorations
- Producing models for the construction of trays for whitening solutions, bites and surgical guides.
- Orthodontic work

- Case study models
- Fabricating removable retainers and splints

## Cautions

Do not use after expiration date.

Do not leave any residual impression material in the sulcus or oral cavity.

Do not swallow impression material! If swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

The product does not normally cause allergic reactions. However, for sensitive persons, an allergic reaction to the product cannot be ruled out.

Use of products containing active sulfur, aluminium chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials, etc.) in conjunction with this product will interfere with the setting reaction of the vinyl polysiloxane material. Use of these materials re-

quires the area to be rinsed thoroughly to remove all residue. Do not use latex gloves.

When taking impressions of areas with severe undercuts and wide interdental spaces, standard blocking-out techniques must be used.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, since the material cannot be removed by dry cleaning.

## Note

To ensure optimal impressions, the temperature of the material should not deviate from 23°C (74°F) before applying. Otherwise, working and setting times will be affected.

## Impression Tray: Preparation and Adhesives

In principle, all common impression trays can be used if a relevant dynamic pressure is guaranteed. When retention is not sufficient, ensure a strong bond to the impression material, by brushing the impression tray with a thin film of adhesive for addition-curing silicone prior to loading the tray with impression material (e.g. Panasil® Adhesive). Allow to dry as per instructions for use and cover with a sufficient amount of impression material.

## Instructions for the Safe Handling of the Kettenbach Plug & Press® System (380 ml Cartridge System; 5:1)

Prior to first use please unscrew the blue plug counter-clockwise and pull it out (Figure 1, Page 88). Check the outlet opening. Then screw on the dynamic mixer (Figure 2, Page 88). Ensure that the positioning aid is on the top.

Please use only the dispensing and mixing components compatible with the Kettenbach 380 ml cartridge system for dispensing and mixing impression material.

The Kettenbach 380 ml cartridge is heavy and could be damaged if dropped. Damaged cartridges must not be used under any circumstances.

When using the material in a Sympress II dispensing device, please activate the speed setting indicated on the product label (Figure 6, Page 89). Failure to comply with this instruction may modify the curing characteristics of the impression materials.

If you are using a unit other than the Sympress mixing device, please follow the instructions for the automatic dispensing and mixing system you are using.

When changing to a different mixing device, always discard at least 3 cm of mixed material.

## Using the Kettenbach Plug & Press® System (380 ml cartridge system; 5:1) in Dispensing Device Sympress/Sympress II

By pressing the plunger return knob, allow the plunger to return to its original position automatically (while the cover is closed; Figure 3, Page 88). Next, open the cover (Figure 4, Page 88).

Insert the Kettenbach 380 ml cartridge with the pre-assembled mixer in the Sympress/Sympress II dispensing and mixing unit (Figure 5, Page 89). If you are working with a previously used cartridge, the filled mixer will already be located on the cartridge as a cap.

If you are using a Sympress II dispensing device, please activate the speed setting indicated on the product label. A light signal will confirm your selection (Figure 6, Page 89).

Close the device cover (Figure 7, Page 89).

### Using the Cartridge for the First Time

Dispense the impression material (ensure that the dynamic mixer is wound into place fully) until the hardener paste and catalyst are mixed homogeneously (see coloring of the mixed impression material); discard at least the first 3 cm when dispensing (Figure 8, Page 89).

At least 3 cm of the mixed material must be discarded each time the plunger is restarted or when using the cartridge in different mixing units.

Load the impression tray with the required amount of material (Figure 9, Page 89) by pressing the appropriate feed key. Hold the impression at an angle when loading and press against it lightly. Leave the mixer in the material when dispensing to avoid air bubbles.

Release the feed key as soon as the required amount of material has been dispensed. The filled dynamic mixer should remain on the cartridge as a seal.

Remove the used mixer before the next application by turning it to the left (Figure 10, Page 89), visually check the outlet opening for blockage and attach a new dynamic mixer (Figure 10, Page 89).

Remove the completely empty jumbo cartridge from the dispensing and mixing unit and dispose of it according to regional regulations.

### Further Use of a Cartridge Already in Use

Wind a new dynamic mixer onto the same mixing unit and begin the new application (material no longer has to be discarded). Proceed with the application as described above.

## **Instructions on the Use of Cartridge Material in the Applyfix 4 Dispensing Gun DS-50 1:1/2:1**

Insert the cartridge into the Applyfix 4 dispensing gun DS-50 1:1/2:1. Ensure that the notches on the cartridge base are pointing down. The clasp will not close if the cartridge is not properly inserted into the dispensing gun.

Remove the cartridge cap. The cap can be replaced after initial use.

Advance plunger into cartridge and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a mixing tip on the cartridge and turn cap 90 degrees in a clockwise direction to lock in place.

Dispense the necessary amount of impression material directly into the impression tray.

Replace the plug after use. Before using the cartridge again, remove the cartridge cap or the filled mixing tip and check the cartridge outlet openings to ensure that no polymerized materials have sealed the openings. Clear any clogged openings and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a new mixing tip and continue as described.

## **Cleaning and Disinfection of the Applyfix 4**

Remove the cartridge.

Remove any uncured paste using a cloth soaked in alcohol.

The Applyfix 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

### **Storage Conditions**

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

### **Disinfection**

Impressions can be disinfected using for example, a 2 % acidic glutaraldehyde solution. Use disinfection solutions specific for impression materials. See manufacturer instructions.

### **Electroplating**

Impressions may be silver-plated with an alkaline-plating solution only.

## Model Preparation

Impressions with Silginat® and Silginat® Strawberry may be poured immediately after disinfection of the impression up to several weeks with standard dental stone (class III–V). Impressions can also be poured several times. A surfactant solution is not required.

## Disposal

The empty cartridges/mixing tips/applicator tips and set material can be disposed in the household rubbish.

## Trademarks

Silginat® and Panasil® are registered trademarks of Kettenbach GmbH & Co. KG.

## Note

Selected Kettenbach materials are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

## Limitation of liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development.

However, users must comply with and consider all recommendations and information in connection with any use.

Updated: October 6, 2020

## Product Overview

Product name	ISO 4823	Consistency (approx.) mm	Mixing ratio and total content volume per unit	Mixing technique	Mixing element
Silginat®	Type 2, Medium bodied	33	5:1, 380 ml cartridge	Sympress / Sympress II or other automatic dispensing and mixing unit	Dynamic mixer, blue
Silginat®	Type 2, Medium bodied	35	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Silginat® Strawberry	Type 2, Medium bodied	33	5:1, 380 ml cartridge	Sympress / Sympress II or other automatic dispensing and mixing unit	Dynamic mixer, blue
Silginat® Strawberry	Type 2, Medium bodied	35	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm

## Technical Data

Product name	Mixing ratio	Working time at 23°C/74° F ≤	Intraoral setting time at 35°C/95°F ≥	Total setting time* ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Elastic-Recovery test (approx.) %	Strain in compression (approx.) %
Silginat®	5:1	1 minute 30 seconds	1 minute 30 seconds	3 minutes	A 45	- 0.20	99.5	5
Silginat®	1:1	1 minute 15 seconds	1 minute 15 seconds	2 minutes 30 seconds	A 42	- 0.20	99.5	7.2
Silginat® Strawberry	5:1	1 minute 30 seconds	1 minute 30 seconds	3 minutes	A 45	- 0.20	99.5	5
Silginat® Strawberry	1:1	1 minute 15 seconds	1 minute 15 seconds	2 minutes 30 seconds	A 42	- 0.20	99.5	7.2

\* Total setting time (removal time from mouth) from start of mix

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