KETTENBACH DENTAL

Simply intelligent

MUCOPREN® SOFT

PATIENT BROCHURE

Patienteninformation

L'information des patients

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DEAR PATIENT,**English**

the bottom of your denture has been fitted with a permanently soft lining to make it as comfortable as possible for you to wear. The soft lining does not smell or taste at all.

Please do not eat for the first two hours or clean your denture by hand, e.g. with a toothbrush, for the first day. In individual cases, the consumption of coffee, carrot juice and red wine might cause discolorations of the denture.

You can help make the soft lining last longer by caring for it carefully. Rinse your denture daily in running water before cleaning it with any established brand of denture cleanser based on active oxygen (e.g. Blend-a-dent 2-Phase, Corega Tabs or Kukident). This reduces colonization by microorganisms. Then carefully clean your denture with a soft toothbrush and your usual toothpaste. Substances that do contain alcohol are not suitable for cleaning your dentures as they can have an adverse effect on the permanently soft lining.

Once you have cleaned your denture, rinse your mouth thoroughly and replace the denture in your mouth. Alternatively, keep your denture in water or a dry place – depending on what your dentist advises. In any case, follow your dentist's advice.

Should parts of the soft lining become detached due to overloading, your dentist can replace them.

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