DEUTSCH

Do not swallow material. If the material is inadvertently swallowed: Consult a medical doctor if problems arise or persist.

Avoid contact with skin and mucous tissue. Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Take the appropriate protective measures (e.g., wear safety glasses). Consult a medical doctor if problems arise or persist.

Safety data sheets are available at www.kettenbach.com.

Application Instructions
After trying-in the restoration – according to the specifications in the instructions for use of Visalys® CemCore – the shade effect of the selected shade can be simulated with the compatible Visalys® CemCore Try In Paste.

In order that the natural shade effect of the core is preserved, the core surface should not be dried.

1. Selection of the required shade and application of the try-in paste with the application tip attached on the inside of the restoration. Dispose of used application tip and replace the sealing cap of the Visalys® CemCore Try In Paste.

2. Placement of the restoration on the core, the cavity or the intended total surface for veneers. Do not apply high pressure during placement to avoid possible breakage of the restoration.

3. Remove excess Visalys® CemCore Try In Paste using a suitable instrument (e.g. brush, cotton pellet). The best simulation of the shade effect is attained if the layer thickness of the try-in paste corresponds to the subsequent thickness of the cement layer.

4. Assessment of the shade effect. Select another shade, as appropriate.

5. Loosen the restoration and clean the adhesive surfaces of the core and the restoration with water. Attention has to be paid to complete removal of the try-in paste to achieve a good adhesive bond for the permanent cementation.

6. Cementation of the restoration with Visalys® CemCore in the selected shade according to the instructions for use of the product.

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Constituenten
The main components of the Visalys® CemCore Try In Paste are polyethylene glycol and silicic acid.

Disposal
The empty syringes and applicator tips can be disposed in the household rubbish.

Trademarks
Visalys® CemCore Try In Paste and Visalys® CemCore are registered trademarks of Kettenbach GmbH & Co. KG.

Note
Selected Kettenbach materials are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist. If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability
Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use of the product.

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ENGLISH

Product Description / Indication
Visalys® CemCore Try In Paste serves to simulate the shade effect of the selected Visalys® CemCore shade on ceramic restorations and for covering the cement joint to avoid a layer of inhibition. The water-soluble paste corresponds in shade and translucence with the completely cured Visalys® CemCore.

Visalys® CemCore Try In Paste is offered in the shades Universal (A2/A3), Dark (A4), Opaque, Bleach and Translucent. Note: Only once the cementation composite Visalys® CemCore is fully polymerized and has attained its final shade, the shade matching with the Visalys® CemCore Try In Paste is attained.

Visalys® CemCore Try In Paste only corresponds to the shades of Visalys® CemCore.

Indications for Use
• To check the shade selected prior to permanent cementation with Visalys® CemCore
• For covering the cement joint to reduce a layer of inhibition

Patient Target Group
Patients being treated in the context of a dental procedure.

Contraindications
Known allergies to the constituents of Visalys® CemCore Try In Paste.

Warnings
Allergic reactions to the product cannot be ruled out in sensitive persons. As a result of the hygroscopic property of the product, fluid shifts can arise in the dentin tubules and cause brief pain sensations in sensitive patients. Desensitizers can be used, as appropriate, or anesthesia performed.

Residues in Visalys® CemCore Try In Paste on the surfaces of the restoration or the core can impair the adhesive strength of Visalys® CemCore. Visalys® CemCore Try In Paste does not set and is not suitable for cementation of restorations.

Cautions
Do not use after the expiration date.

Do not swallow material. If the material is inadvertently swallowed: Consult a medical doctor if problems arise or persist.

Avoid contact with skin and mucous tissue.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Take the appropriate protective measures (e.g., wear safety glasses). Consult a medical doctor if problems arise or persist.

Safety data sheets are available at www.kettenbach.com.

Application Instructions
After trying-in the restoration – according to the specifications in the instructions for use of Visalys® CemCore – the shade effect of the selected shade can be simulated with the compatible Visalys® CemCore Try In Paste.

In order that the natural shade effect of the core is preserved, the core surface should not be dried.

1. Selection of the required shade and application of the try-in paste with the application tip attached on the inside of the restoration. Dispose of used application tip and replace the sealing cap of the Visalys® CemCore Try In Paste.

2. Placement of the restoration on the core, the cavity or the intended total surface for veneers. Do not apply high pressu- re during placement to avoid possible breakage of the restoration.

3. Remove excess Visalys® CemCore Try In Paste using a suitable instrument (e.g. brush, cotton pellet). The best simulation of the shade effect is attained if the layer thickness of the try-in paste corresponds to the subsequent thickness of the cement layer.

4. Assessment of the shade effect. Select another shade, as appropriate.

5. Loosen the restoration and clean the adhesive surfaces of the core and the restoration with water. Attention has to be paid to complete removal of the try-in paste to achieve a good adhesive bond for the permanent cementation.

6. Cementation of the restoration with Visalys® CemCore in the selected shade according to the instructions for use of the product.

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